



DATA SHEET

Large Volume Pump (LVP)

Ivenix Infusion System

Compatible Cleaning Solutions and Disinfectants

Proper care and maintenance of the Ivenix Large Volume Pump (LVP) will help ensure optimal performance and longevity of the device. Fresenius Kabi recommends that the LVP is cleaned and sanitized between patient use and prior to maintenance. The following is a list of recommended and compatible cleaning solutions and disinfectants.

Cleaning Solutions

- EcoLab® Asepti-Zyme™ Multi-Enzymatic Detergent
- ENZOL® Enzymatic Detergent
- Metrex™ Metrizyme™ Dual-Enzymatic Detergent
- STERIS® Valsure® Alkaline Detergent
- STERIS Valsure Neutral Detergent

Disinfectants

- Metrex CaviCide™ *
- Clorox® Regular Bleach2 *
- Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipe *
- PDI® Sani-Cloth® Bleach Germicidal Disposable Wipe *
- PDI Super Sani-Cloth Germicidal Disposable Wipe *
- Wexford Innovations Ready To Use Thymo-Cide

Note:

- Follow manufacturer's instructions on dilutions (if concentrate) and on recommended dwell times.
- Consult the Instructions For Use (IFU) for instruction and cautions for cleaning.
- Cleaning solutions and disinfectants effective against SARS-CoV-2 are indicated by an asterisk (*).

For questions, contact info@ivenix.com.

Distributed in the U.S. by
Fresenius Kabi USA, LLC.
Three Corporate Drive
Lake Zurich, IL 60047
Phone: 1.888.386.1300
www.fresenius-kabi.us/ivenix

Manufactured for: Fresenius Kabi, AG
61352 Bad Homburg, Germany

 **FRESENIUS
KABI**
caring for life